

The new Cutera®
Titan® gives you an
all natural option

to actively turn back the clock
on aging skin. Using light
energy to stimulate new
collagen growth deep
beneath your skin's surface,
Titan can tighten skin on your
face, arms, abdomen, and
legs. It's an ideal solution to
enhance your youthful and
healthy appearance without
surgery, recovery time,
or injections.

titan®

SKIN
TIGHTENING

CUTERA™

WORLD HEADQUARTERS
3240 Bayshore Boulevard, Brisbane, CA 94005
Phone: 1-888-4CUTERA | www.cutera.com

©2012 Cutera, Inc. All rights reserved. Titan and Cutera
are registered trademarks of Cutera, Inc.

AP0287 Rev A (04/12)

CUTERA™

Skin Tightening with Titan

How does the Cutera Titan work?

Titan utilizes a safe, infrared light to heat the dermis well below the skin's surface. This heating can cause immediate collagen contraction resulting in tighter skin. During the procedure, the skin's surface (epidermis) is protected through continuous cooling with the Titan handpiece. After the procedure, new collagen growth results in further tightening with the improvement of lax skin, folds and lines.

The Titan Difference

Titan employs light energy because it provides a more uniform distribution of energy for heating the deep dermis than other energy sources, such as RF (radiofrequency).

What areas respond best to the Titan procedure?

Patients have been successfully treated on many areas of the body where skin is lax. Loose skin along the jaw line, under the chin, the abdomen and upper arm have responded particularly well to the procedure.

Who can be treated?

Since the Titan procedure is heating tissue under the skin's surface, patients of any skin type can be treated. Men and women of all ages and skin types are potential candidates. Your physician can advise you if the Titan is right for you.

How many treatments will I need?

A single treatment has produced very good results for some patients. Your physician will advise you as to the number and frequency of treatments recommended for optimal results.

What happens after the treatment? Any side effects?

Immediately following the treatment, you may notice some redness and mild swelling in the treated area. Both should dissipate within a few hours. Consult with your medical professional for complete information on the benefits and risks of treatment.

When will I see results?

Some patients see results immediately. However, for most patients, the results are gradual over a three to six month period as new collagen is formed and the skin tightens.

What do treatments feel like?


With each Titan pulse, you will feel a brief heating sensation. Some patients are more comfortable taking a mild pain reliever beforehand. Many patients are comfortable without anything. Your physician will advise you.

Has the Titan been tested and approved?

The Titan has been carefully studied in clinics around the world. It is cleared for the treatment of wrinkles in Europe and for general dermatologic use in Canada. In the US, it is cleared for topical heating to increase local tissue temperatures and circulation. An additional 510(k) clearance is pending in the US for the treatment of wrinkles.



Before

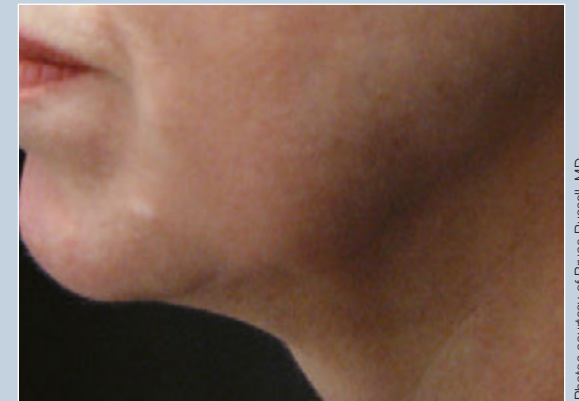

After 2nd Treatment

Photos courtesy of Bruce Russell, MD

Schedule your consultation now to find out if you are a good candidate for the Titan procedure.

www.my-face-aesthetics-clinic.co.uk
58 St Georges Road, Bolton BL1 2DD
Tel: 01204 866880